Directions for Lay Users

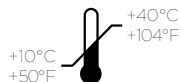
en | da | de | es | fi | fr | it | ja | nl | no | pl | pt | ru | sv

SDM

Operation Temperature

EN Sentec Digital Monitor / Docking Station
DA Sentec digital monitor / Docking station
DE Sentec Digital Monitor / Docking Station
ES Monitor digital Sentec / Estación de calibración
FI Sentec Digital Monitor / Telakka-asema
FR Sentec Digital Monitor / Station d'accueil
IT Sentec Digital Monitor / Docking Station

JA センテックデジタルモニタ / ドッキングステーション
NL Sentec Digital Monitor / Docking Station
NO Sentec Digital Monitor / Dokkingstasjon
PL Monitor cyfrowy Sentec / Stacja dokująca
PT Monitor digital Sentec / Docking Station
RU Цифровой монитор Sentec / док-станция
SV Sentec digital monitor / Dockningsstation


VS-A/P/N

EN V-Sign™ Sensor 2 ("Sensor")
DA V-Sign™-sensor 2 ("sensor")
DE V-Sign™ Sensor 2 („Sensor“)
ES Sensor V-Sign™ 2 ("Sensor")
FI V-Sign™-sensori 2 ("sensori")
FR Capteur V-Sign™ 2 (« Capteur »)
IT Sensore V-Sign™ 2 ("Sensore")

JA V-Sign™センサ2 (「センサ」)
NL V-Sign™ Sensor 2 („Sensor“)
NO V-Sign™-sensor 2 ("sensor")
PL Czujnik 2 V-Sign™ („czujnik“)
PT Sensor V-Sign™ 2 ("sensor")
RU Датчик V-Sign™ 2 ("sensor")
SV V-Sign™ Sensor 2 ("sensor")

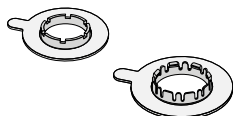
MAR/MARe - MI/SF

EN Multi-Site Attachment Ring
DA fikseringsring
DE Befestigungsring
ES Anillo de fijación multizona
FI Kiinnitysrengas
FR Anneaux de fixation
IT Anello adesivo

JA マルチサイト・アタッチメントリング
NL Bevestigingsring
NO Festering
PL Pierścień mocujący
PT Anel de fixação multilocalização
RU Крепежное кольцо
SV Multi-Site fixeringsring

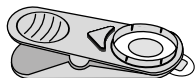
EC-MI

EN Ear Clip
DA Øreclip
DE Ohrclip
ES Clip de oreja
FI Korvaklipsi
FR Pince d'oreille
IT Clip Auricolare

JA ヤークリップ
NL Oorclip
NO Øreklips
PL Klips nauszny
PT Clípe de orelha
RU Ушной зажим
SV Öronklämma

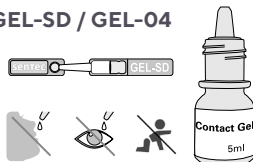GEL-SD / GEL-04


EN Contact Gel
DA Kontaktgel
DE Kontaktgel
ES Gel de contacto
FI Kontaktigeeli
FR Gel de contact
IT Gel di Contatto

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
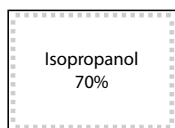
EN | Directions for Lay Users

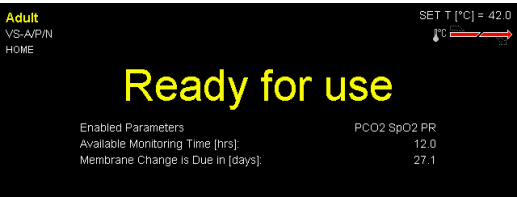A) Sensor application using MAR/e-MI or MAR/e-SF

ENSURE SDM DISPLAYS
“READY FOR USE”.

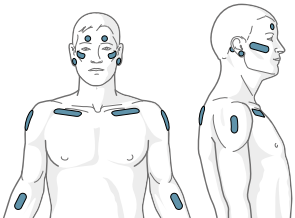- 1 Clean Measurement Site selected by the home care provider, using Isopropyl 70 % soaked wipe.
- 2 Pull adhesive tape off Multi-Site Attachment Ring (MAR/e) and attach MAR/e to measurement site, pressing gently around the ring.
- 3 Apply 1-2 drops of Contact Gel to the skin area in the center of the ring.
Note: Avoid contact with injured skin. Do not use when exhibiting allergic reactions to one of the components.
- 4 Open Docking Station, remove sensor, close Docking Station.
Note: Do not touch the sensor surface with any hard objects or fingernails. Otherwise, the sensor could be damaged.
- 5 Click sensor into MAR/e and rotate sensor to spread Contact Gel.
Push gently!
- 6 Secure cable with an adhesive tape.
Important Note: Route the sensor cable properly (i.e., with a Clothing Clip) to avoid entanglement or strangulation.

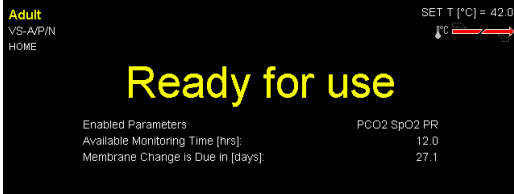
If you need to interrupt a session after monitoring has started (e.g., to go to the bathroom), you may disconnect the sensor for up to 20 minutes.

20 min


If you disconnect the sensor for more than 30 minutes, the recording will be lost.

▶ **WATCH VIDEO**


B) Sensor application using EC-MI

ENSURE SDM DISPLAYS
"READY FOR USE".

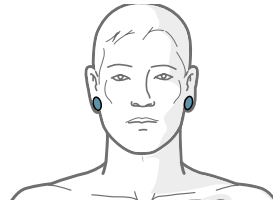MEASUREMENT SITES

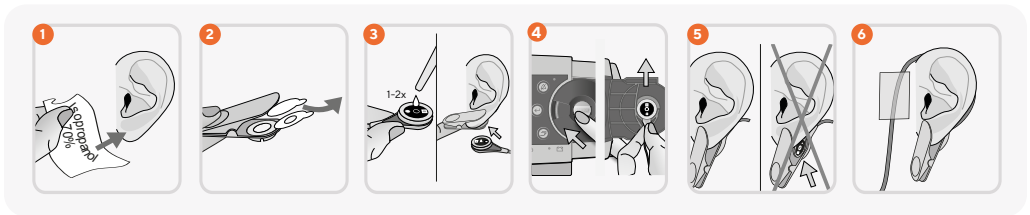1 Clean Measurement Site selected by the home care provider, using Isopropanol 70 % soaked wipe.

2 Pull both adhesive tapes off Ear Clip, attach Ear Clip to earlobe with retainer ring on the back side. Verify that the skin under the retainer ring's adhesive is not wrinkled and that the hole in the center of the retainer ring completely covers the skin.

3 Apply 1-2 drops of Contact Gel to the center of the sensor surface.
Note: Avoid contact with injured skin. Do not use when exhibiting allergic reactions to one of the components.

4 Open Docking Station, remove sensor, close Docking Station.

Note: Do not touch the sensor surface with any hard objects or fingernails. Otherwise, the sensor could be damaged.

5 Click sensor into Ear Clip, rotate sensor to spread Contact Gel.

Note: Earlobe must cover the entire sensor surface. Be sure to hold the sensor such that the contact liquid does not run off the sensor face.

6 Wrap the sensor cable around the ear and secure it with an adhesive tape.

Note: Do not lie on Ear Clip as this may lead to incorrect measurements.

Important Note: Route the sensor cable properly (i.e., with a Clothing Clip) to avoid entanglement or strangulation.

If you need to interrupt a session after monitoring has started (e.g., to go to the bathroom), you may disconnect the sensor for up to 20 minutes.

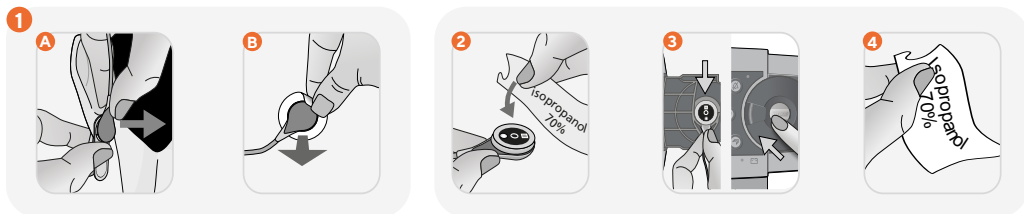
If you disconnect the sensor for more than 30 minutes, the recording will be lost.

▶ WATCH VIDEO

C) Sensor Removal

1 When monitoring is completed, remove **4** Ear Clip or gently peel off **6** Multi-Site Attachment Ring, then remove sensor from retainer ring.

2 Clean sensor and cable with Isopropanol 70 % soaked wipe.

3 Open Docking Station, put sensor into cradle inside the door with the red light visible, close Docking Station.

Note: Do not turn off the device.

4 Clean measurement site with Isopropanol 70 % soaked wipe.

Note: Your home care provider will ensure professional and hygienic maintenance of the system once returned.

! IMPORTANT: Do not turn off the device at the end of the measurement. The evaluation and analysis of the measurements is carried out by the home care provider. Analysis of data is done by the supervising medical doctor.

! IMPORTANT

- The device may only be installed by a home care provider's instructed person.
- In the event of unforeseen incidents, error messages, unexplained changes in the performance of the device, or if you require any other assistance, please contact your home care provider's instructed person. If in doubt, you can interrupt the measurement by removing the sensor, cleaning it and placing it in the Docking Station.
- Solely use the device as per this Lay User Manual. Do not make any changes to the device settings. Sentec is not responsible for any injuries or damages incurred due to improper use of the device contradictory to this Lay User Manual.
- Do not make any changes to the mains plug of the device and secure the mains cable against squeezing or bending.
- Keep electronic devices and cables out of reach of children and pets and supervise them during monitoring.
- Make sure that children do not swallow Contact Gel or small parts, such as the sensor, the Multi-Site Attachment Rings, or Ear Clip.
- Only use Sentec Contact Gel. Do not use any other contact gel/liquid. Do not use Sentec Contact Gel on any open wound, do not swallow it or apply it to the eyes.
- It is not recommended to use the Contact Gel on patients who exhibit allergic reactions to any of the components.
- Temporary skin irritations or erythema could occur during monitoring at increased sensor temperatures (i.e., above the normal body surface temperature). The skin reddening will generally disappear within a few hours to max. a few days without any side effects or scars. The likelihood of these occurrences depends on patient skin condition, sensor temperature and exposure time.
- Do not reattach the sensor to the same site if any skin irritations are noted during site inspection.
- It is not recommended to use sensor attachment accessories on patients who exhibit allergic reactions to adhesive tapes.
- Application of any pressure to the measurement site (e.g., by using a pressure bandage) may cause pressure ischemia at the measurement site and, consequently, inaccurate measurements, necrosis or – in combination with heated sensors – burns.
- The Multi-Site Attachment Rings and Ear Clip are for single use.
- Do not pour any liquid on the SDM, its accessories, connectors, switches, or openings in the chassis. If the SDM has been wetted accidentally, it must be removed from AC power, wiped dry externally, allowed to dry thoroughly, and inspected by qualified service personnel before further use. The SDM is protected against vertically falling drops of water, the Isolation Transformer is protected against splash water.
- The SDMS may not be installed and operated in moist rooms (e.g., bathroom).
- Keep the device away from heat sources (e.g., heating) and at least 50 cm from flammable materials.
- Mobile telephones and other radio communication devices must be kept 1 m away from the device.
- Do not cover the device.
- Do not use a non-invasive blood pressure cuff or other constricting devices on the same limb as the sensor. This may result in inaccurate measurements.
- Protect all devices from lint, dust, and direct sunlight.
- High ambient light conditions may result in inaccurate measurements.

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Sentec AG