	<p>en Membrane Changer de Membranwechsler es Intercambiador de membrana fr Changeur de membrane pt Carregador de membrana it Kit di Sostituzione di Membrana zh 膜更換器 ja メンブランチェンジャー ar مغير الغشاء bg Зареджачо се приспособление за смяна на мембрана cs Membrána Changer da Membranskifter el Εργαλείο αλλαγής μεμβράνης et Membraanivaheti fi Kalvonvaihdin hu Membráncserélő lt Membranų keitiklis lv Membrānu mainītājs nl Membraanwisselaar no Membranskifter pl Zmienarka membrany ro Schimbător de membrană ru Устройство для замены мембраны sh Memjač membrane sv Membranbytare</p>		

<p>1</p>	<p>2</p>
<p>3</p>	<p>4</p>
<p>5</p>	<p>6</p>
<p>7</p>	<p>8</p>

en Refer to the Instruction Manual for the SDMS, Technical Manual for the SDM or the respective Directions for Use for warnings, cautions and additional information such as instructions, routine checks, or maintenance recommendations. These, as well as SDMS related tutorials are available on www.sentec.com/ifu.

Warnings/Cautions Do not change the membrane of SenTec V-Sign™ Sensors or OxiVen™ Sensors by other means than the SenTec Membrane Changer. Otherwise, the sensor may be damaged or an inappropriate membrane application may reduce the accuracy of the measurement. Perform sensor membrane changes under clean working conditions only. Do not touch the sensor membrane with any sharp-edged objects, including your fingernails. Damage of the sensor membrane results in reduced accuracy of the sensor readings. If the sensor membrane is damaged, you must change the membrane. Do not touch the delicate sensing elements on the surface of the sensor if the membrane is missing. Do not sterilize any parts of the MC (e.g. by irradiation, steam, ethylene oxide or plasma method).

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