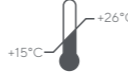
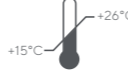
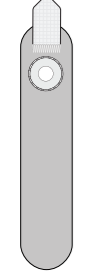
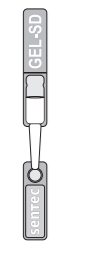
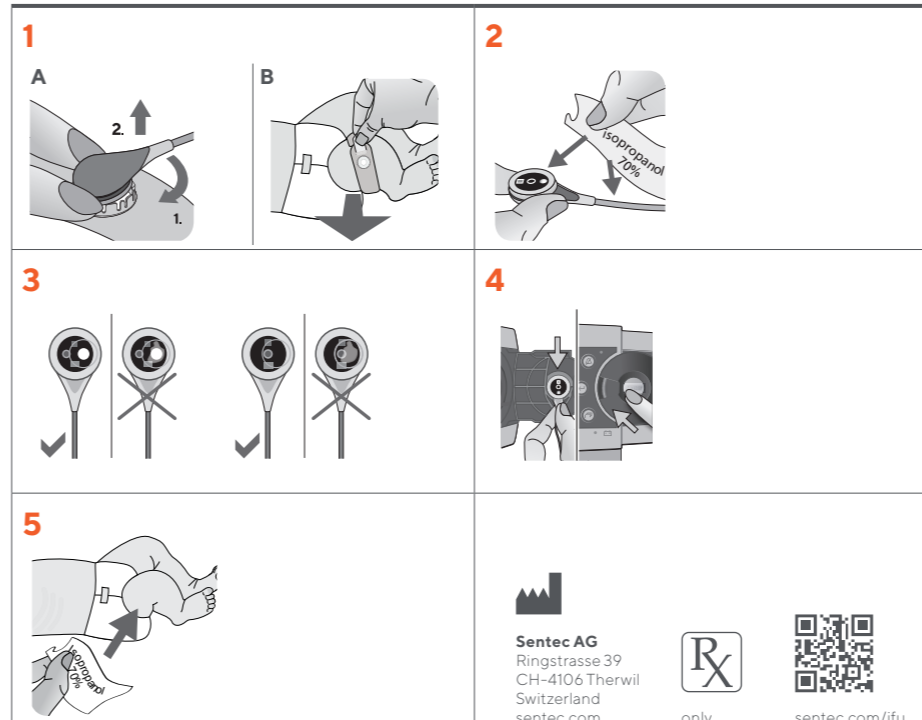
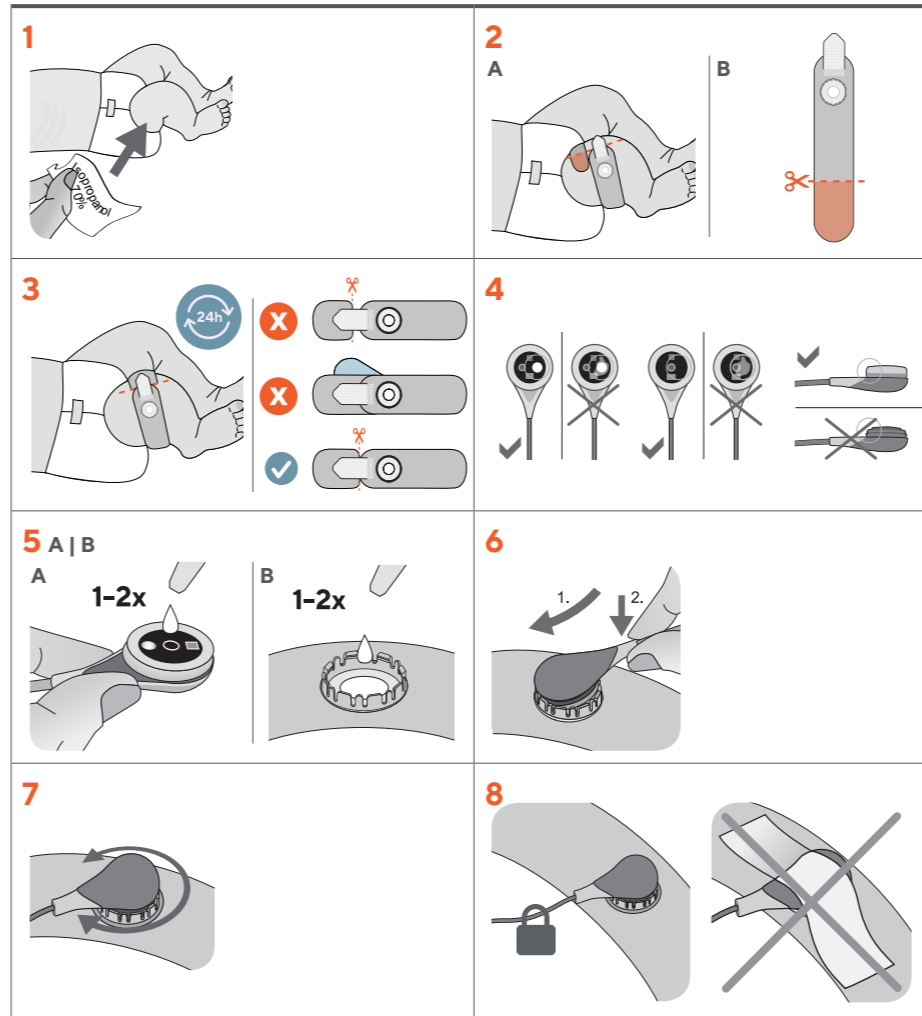
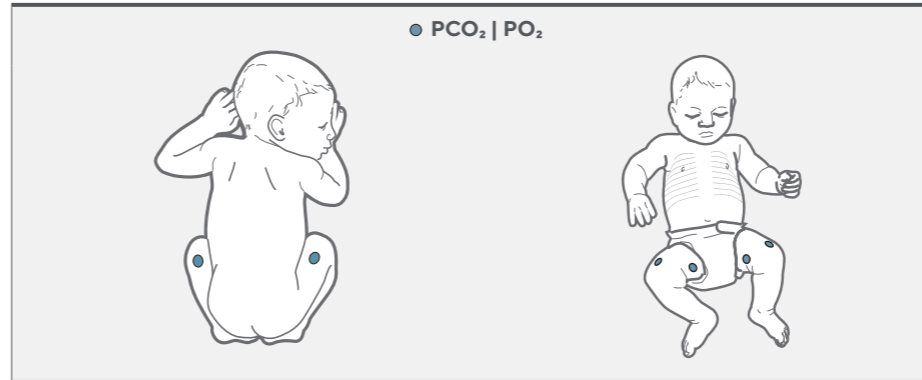
<p>VS-A/P/N* PCO₂ SpO₂/PR</p> 	<p>V-Sign™ Sensor 2 en V-Sign™ sensor 2 de Sensor V-Sign™ 2 fr Capteur V-Sign™ 2 pt Sensor V-Sign™ 2 it Sensore V-Sign™ 2 ja V-Sign™ センサ 2 ar مستشعر V-Sign™ 2 bg Сензор V-Sign™ 2 cs Senzor V-Sign™ 2 da V-Sign™-sensor 2 el Αισθητήρας V-Sign™ 2 et V-Sign™-iandur 2 fi V-Sign™-sensor 2 hu V-Sign™-érzékelő 2 lt Jutiklis V-Sign™ 2 lv Sensors V-Sign™ 2 nl V-Sign™ sensor 2 no V-Sign™-sensor 2 pl Czujnik 2 V-Sign™ ro Senzor V-Sign™ 2 ru Датчик 2 V-Sign™ sh V-Sign™ sensor 2 sv V-Sign™ sensor 2</p>	<p>IPX7</p>   
<p>OV-A/P/N* PCO₂ PO₂ SpO₂/PR</p> 	<p>OxiVent™ Sensor en OxiVent™ Sensor de Sensor OxiVent™ fr Capteur OxiVent™ pt Sensor OxiVent™ it Sensore OxiVent™ ja OxiVent™ センサ ar مستشعر OxiVent™ bg Сензор OxiVent™ cs Senzor OxiVent™ da OxiVent™-sensor el Αισθητήρας OxiVent™ et OxiVent™-iandur fi OxiVent™-sensori hu OxiVent™-érzékelő lt Jutiklis OxiVent™ lv Sensors OxiVent™ nl OxiVent™-sensor no OxiVent™-sensor pl Czujnik OxiVent™ ro Senzor OxiVent™ ru Датчик OxiVent™ sh OxiVent™ sensor sv OxiVent™ sensor</p>	<p>IPX7</p>   
<p>NON-ADHESIVE WRAP</p> 	<p>Non-Adhesive Wrap en Klettbandapplikator de Cinta no adhesiva fr Sangle non adhésive pt Faixa não adesiva it Fascia non adesiva ja 非粘着式バンド ar حزام غير لاصق bg Headzivna лента cs Nepřilnavý pásek da Ikke-klebede bånd el Μη κολλητική ταινία πρόσδεσης et Mittekleevuv kinnituspael fi Tarttumaton kääre hu Téporázás pánt li Neilipõa viituras lv Neļipõa lente nl Niet-klevende manchet no Ikke-adhesivt omslag pl Opaska nieprzyklepna ro Bandaj neadeziv ru Headzivnaya lentna sh Neprilnajuća traka sv Tejprfritt förband</p>	  
<p>GEL-SD*</p> 	<p>Contact Gel en Kontaktgel de Gel de contacto fr Gel de contact pt Gel de contato it Gel di contatto ja コンタクトジェル ar جل موصول bg Контактен гел cs Kontaktní gel da Kontaktgel el Γέλη επαφής et Kontaktgeel fi Kontaktigeeli hu Kontaktgél lt Kontaktinis gelis lv Kontaktzēleja nl Contactgel no Kontaktgel pl Żel przewodzący ro Gel de contact ru Контактный гель sh Kontaktini gel sv Kontaktgel</p> <p>Composition de Zusammensetzung es Componentes fr Constituants pt Componentes it Componenti ja コンポーネント ar المكونات bg Компоненти cs Součásti da Komponenter el Συστατικά στοιχεία et Osad fi Osat hu Komponensek lt Komponentai lv Sastāvdaļas nl Componenten no Komponenter pl Składniki ro Componente ru Компоненты sh Komponente sv Komponenter</p> <p>(~90%) H₂O, 12-Propanediol (C₃H₈O₂), Glycerol (C₃H₈O₃)</p>	 
<p>GEL-04*</p> 	<p>Contact Gel en Kontaktgel de Gel de contacto fr Gel de contact pt Gel de contato it Gel di contatto ja コンタクトジェル ar جل موصول bg Контактен гел cs Kontaktní gel da Kontaktgel el Γέλη επαφής et Kontaktgeel fi Kontaktigeeli hu Kontaktgél lt Kontaktinis gelis lv Kontaktzēleja nl Contactgel no Kontaktgel pl Żel przewodzący ro Gel de contact ru Контактный гель sh Kontaktini gel sv Kontaktgel</p> <p>Composition de Zusammensetzung es Componentes fr Constituants pt Componentes it Componenti ja コンポーネント ar المكونات bg Компоненти cs Součásti da Komponenter el Συστατικά στοιχεία et Osad fi Osat hu Komponensek lt Komponentai lv Sastāvdaļas nl Componenten no Komponenter pl Składniki ro Componente ru Компоненты sh Komponente sv Komponenter</p> <p>(~90%) H₂O, 12-Propanediol (C₃H₈O₂), Glycerol (C₃H₈O₃)</p>	 

en Please consult the Instruction Manual of the Sentec Digital Monitoring System for complete descriptions of warnings/cautions, recommendations of temperature setting and site time combinations, and additional information.

Warnings/Cautions Ensure that the Non-Adhesive Wrap is correctly measured around the thigh to avoid a fit that is too loose or tight. Incorrect measurement can cause inaccurate tPCO₂ readings. Avoid overlap between the two ends of the wrap and skin contact with the closure tab. Check the site after sensor application to ensure adequate circulation. A circulation check should be included with each assessment. Avoid damage to the sensor caused by sharp objects or fingernails. Do not use sensor attachment accessories

or Contact Gel on injured skin or on patients who exhibit allergic reactions. The use of temperatures higher than 41° C requires special attention to patients with susceptible skin, e.g. neonates. A flat, well-perfused area of intact skin is needed. Do not place over large superficial veins or areas of skin breakdown or edema. Do not use product for longer than 24 hours. Do not re-attach the sensor to the same measurement site if any skin irritations are noted. Do not remove the membrane of the sensor for cleaning and disinfection. Use 70% Isopropanol (for external use) or approved cleaning agent listed on www.sentec.com/ifu. Do not clean the sensor in an ultrasonic cleaner. Do not sterilize any parts described herein. Do not immerse the connector of the sensor cable in any liquid solution.

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Sentec AG