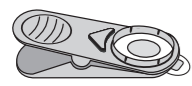
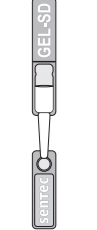
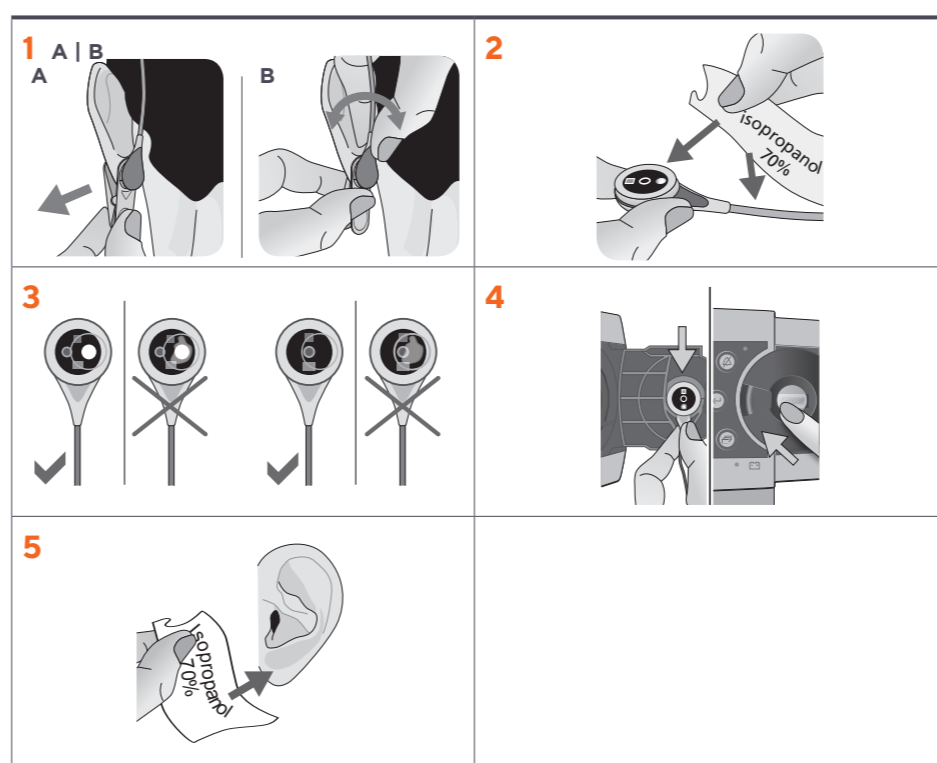
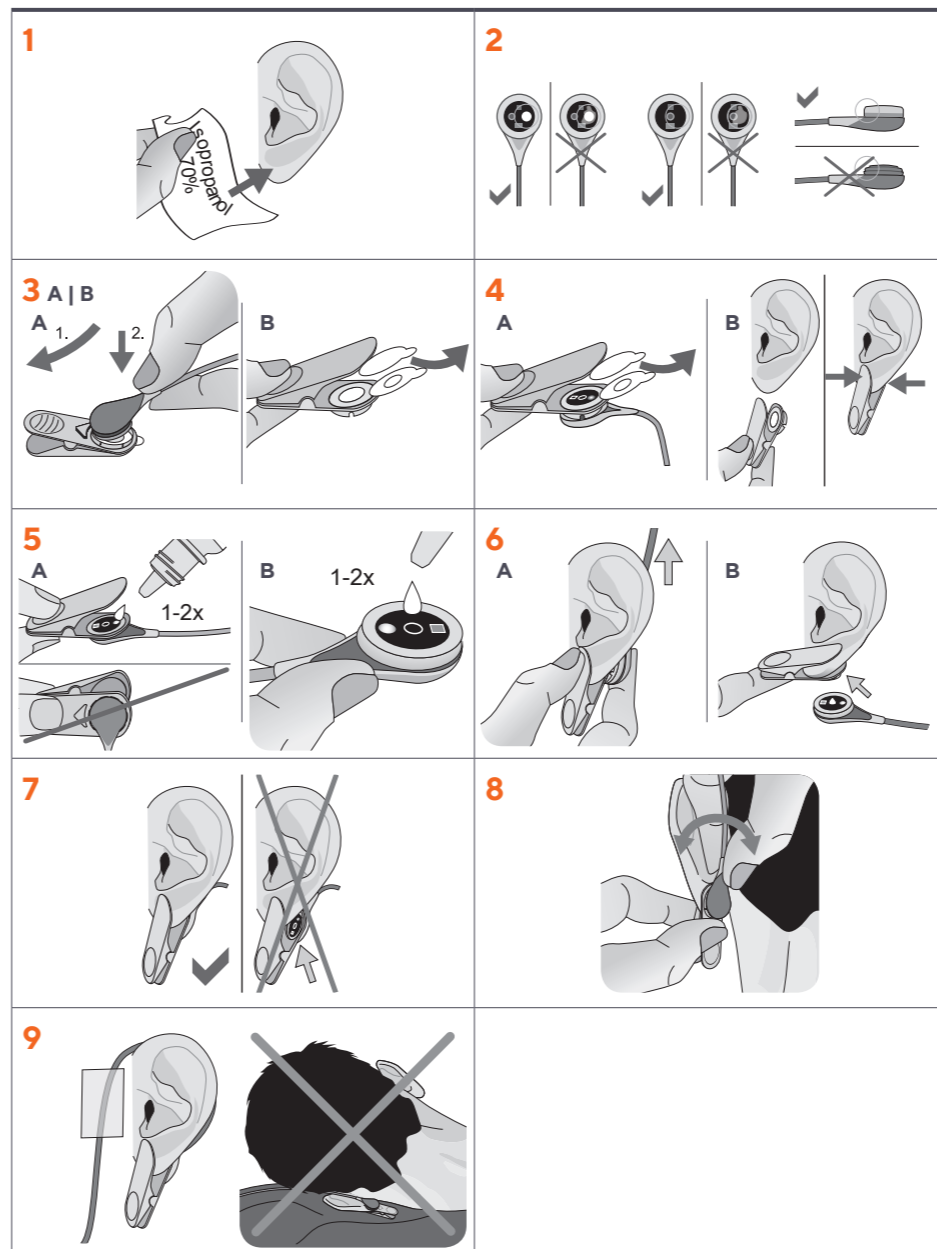
<p>VS-A/P/N * PCO₂ SpO₂/PR</p> 	<p>en V-Sign™ Sensor 2 de V-Sign™ Sensor 2 es Sensor V-Sign™ 2 fr Capteur V-Sign™ 2 pt Sensor V-Sign™ 2 it Sensore V-Sign™ 2 zh OxiVent™ 传感器 ja V-Sign™ センサ ar مستشعر V-Sign™ 2 bg Сензор V-Sign™ 2 cs Senzor V-Sign™ 2 da Adbittmapp V-Sign™ 2 el V-Sign™-i andur 2 fi V-Sign™-sensori 2 hu Jutiteles V-Sign™ 2 lv Sensors V-Sign™ 2 nl V-Sign™ Sensor 2 no V-Sign™-sensor 2 pl Czujnik 2 V-Sign™ ro Senzor V-Sign™ 2 ru Датчик 2 V-Sign™ sh V-Sign™ sensor 2 sv V-Sign™ sensor 2</p>	<p>IPX7</p>  <p>+15°C  +26°C</p> 
<p>OV-A/P/N * PCO₂ PO₂ SpO₂/PR</p> 	<p>en OxiVent™ Sensor de OxiVent™ Sensor es Sensor OxiVent™ fr Capteur OxiVent™ pt Sensor OxiVent™ it Sensore OxiVent™ zh OxiVent™ 传感器 ja OxiVent™ センサ ar مستشعر OxiVent™ bg Сензор OxiVent™ cs Senzor OxiVent™ da OxiVent™-sensor el Adbittmapp OxiVent™ et OxiVent™-i andur fi OxiVent™-sensori hu OxiVent™-érzékelő it Jutiteles OxiVent™ lv Sensors OxiVent™ nl OxiVent™ Sensor no OxiVent™-sensor pl Czujnik OxiVent™ ro Senzor OxiVent™ ru Датчик OxiVent™ sh OxiVent™ sensor sv OxiVent™ sensor</p>	<p>IPX7</p>  <p>+15°C  +26°C</p> 
<p>EC-MI *</p> 	<p>en Ear Clip for mature, intact skin de Ohrclip für normale, gesunde Haut es Clip de oreja para piel madura e intacta fr Pince d'oreille pour les peaux intactes pt Clip de Orelha para pele madura e intacta it Clip Auricolare per pelli mature e intatte zh 耳夹 適合正常、健康的皮肤 ja 耳夾 クリップ 健康な肌用 ar حذاء الأذن للجلد السليم المتقنة في السن bg Клипс за ухо за зряла, здрава кожа cs Klip na ucho pro zralou, nerozkroženou pokožku da Ohrclip til moden, ubeskadiget hud el Κλιπ οτιού για ώριμο, άβηκτο δέρμα et Kõrvaklamber normaalise, terve nahaj jaoks fi Korvaklipsi ehjälle iholle hu Füleklipetű éretti, ép bőrre lt Ausies spaustukas normaliai, sveikai odai lv Ausis klipsis nobriedušai ādai bez bojājumiem nl Oorclip voor volgreede, intacte huid no Øreklipp til moden, intakt hud pl Klips nauszny do skóry dojrzałej, nienaruszonej ro Clip de urechi pentru pielea matură și intactă ru Ушная клипса для зрелой, неповрежденной кожи sh Klips za uvo za zrelo, neostetenu kožu sv Öronklamma för mogen, intakt hud</p>	<p>+10°C  +30°C</p> <p>25% </p> 
<p>GEL-SD *</p> 	<p>en Contact Gel de Kontaktgel es Gel de contacto fr Gel de contact pt Gel de contato it Gel di Contatto zh 接觸凝膠 ja コンタクトジェル ar جل موصول bg Контактен гел cs Kontaktní gel da Kontaktgel el Γέλη επαφής et Kontaktgeel fi Kontaktigeeli hu Kontaktgel it Kontaktinis gelis lv Kontaktzēģeļa nl Contactgel no Kontaktgel pl Żel przewodzący ro Gel de contact ru Контактный гель sh Kontaktini gel sv Kontaktgel</p>	<p>Composition de Zusammensetzung es Componentes fr Constituants pt Componentes it Componenti zh 组件 ja コンポーネント ar المكونات bg Компоненти cs Součásti da Komponenter el Συστατικά στοιχεία et Osad fi Osat hu Komponensek it Componenti lv Sastāvdaļas nl Componenten no Komponenter pl Składniki ro Componente ru Компоненты sh Komponente sv Komponenter</p> <p>+10°C </p> 
<p>GEL-04 *</p> 	<p>(>90%) H₂O, 1,2-Propanediol (C₃H₈O₂), Glycerol (C₃H₈O₃)</p>	<p>+10°C </p> 

* The components listed above do not necessarily correspond to the scope of delivery.

en Please consult the Instruction Manual of the Sentec Digital Monitoring System for complete descriptions of warnings/cautions, recommendations of temperature setting and site time combinations, and additional information.

Contraindications
The use of the Ear Clip is contraindicated in case of:
- patients whose earlobes are too small to ensure adequate sensor application (e.g. neonates).

Warnings/Cautions
The use of temperatures higher than 41°C requires special attention to patients with susceptible skin, e.g. neonates, geriatric patients, burn victims, patients with skin diseases. Ensure that all parts are applied correctly and physically intact and that the sensor is not damaged by sharp-edged objects or fingernails. Incorrect application and damage can cause incorrect measurements. Always apply sensor and Contact Gel on intact and closed skin sites. Pierced earlobes may result in incorrect tPCO₂ and/or tPCO₂ measurements. Do not apply on patients who exhibit allergic reactions to these disposables. Do not use the

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Sentec AG