

# tCOM+ Transcutaneous Monitoring System



## Monitor Overview



### 1. Favorites

Build your own library of frequently used workflows by saving them to your favorites, accessible here.



### 2. Monitoring Time

Indicates the remaining measurement time (h).

**Yellow:** sensor calibration is recommended

**Yellow with cyan background:** sensor calibration is mandatory



### 3. Heating Mode

Indicates current sensor temperature and special temperature settings.



### 4. Patient Type

Indicates Adult or Neonatal mode



### 5. Sensor Maintenance

**Grey:** Membrane change due in >3 days

**Yellow:** Membrane change due in <3 days

**Yellow with cyan background:** Membrane change required



### 6. Alarm bar

Displays active alarms and messages.



### 7. Alarm

Silence active alarms.



### 8. Gas Level

View the calibration gas bottle fill state (%).



### 9. Screenshot

Take a screenshot of the current screen.



### 10. Battery

View battery status and capacity.



### 11. Main Menu

Access sub-menus to adjust alarm limits and measurement settings, view tutorials, add events, and view and change profiles.



### 12. Profile

Indicates the selected profile and monitoring parameter(s).



### 13. Monitor Status

Indicates current monitor status, Calibration in Progress or Ready for use.

## Main Menu Overview



### Alarms

Set and adjust parameter alarm limits



### Audio

Adjust alarm volume



### Measurement Settings

Adjust parameters, site time, and temperature



### Trend Settings

Modify trend range and trend time scale



### Events

Log interventions as events



### Tutorials

View instructions for common workflows



### Sensor Maintenance

Calibrate sensor, confirm membrane change, and initiate a Sensitivity Test



### Profile Selection

Select preconfigured measurement profiles



### Baselines

View and set baselines



### Display

Adjust brightness and enable sleep mode



### Review & Export

View and export screenshots and data



### Advanced Settings

Edit profiles and adjust system settings




### System Information

View monitor and sensor-related information

## Before Monitoring: Inspect the Sensor



## Touch Gestures

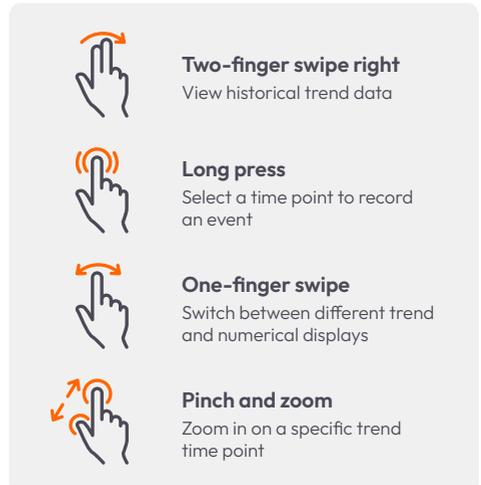

VISIT PRODUCT SUPPORT



Sentec AG  
Ringstrasse 39  
4106 Therwil  
Switzerland

MMQ-251-V1 | 2024-08

sentec.