
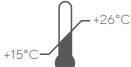
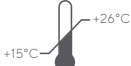
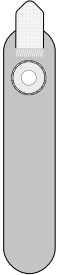
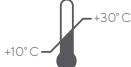
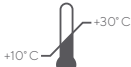
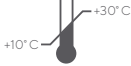
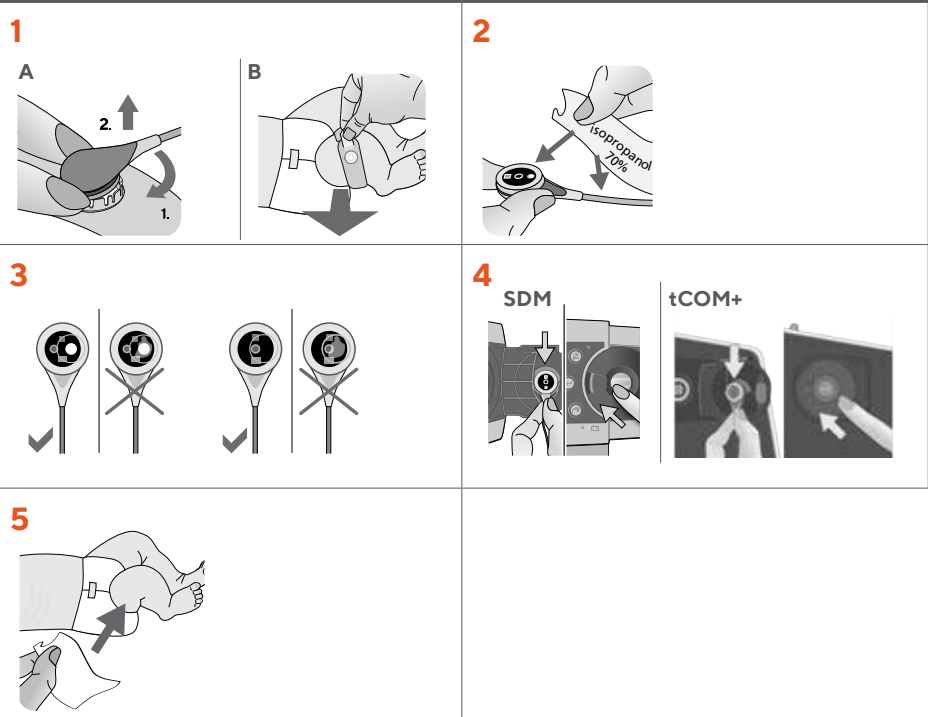
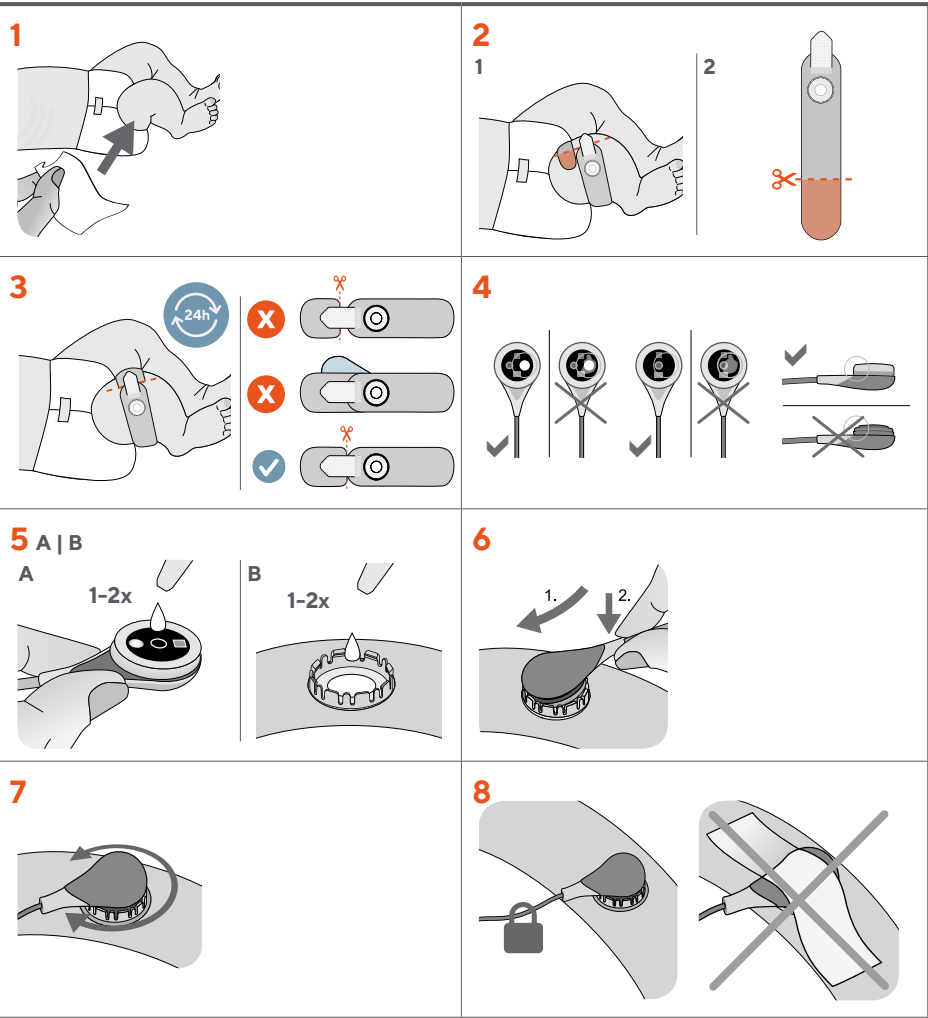
VS-A/P/N * PCO₂ SpO₂/PR 	en V-Sign™ Sensor 2 es Sensor V-Sign™ 2 fr Capteur V-Sign™ 2 pt Sensor V-Sign™ 2 it Sensore V-Sign™ 2 ja V-Sign™ センサ 2 ar مستشعر V-Sign™ 2 bg Сензор V-Sign™ 2 cs Senzor V-Sign™ 2 da V-Sign™-sensor 2 el Αισθητήρας V-Sign™ 2 et V-Sign™-i andur 2 fi V-Sign™-sensori 2 hu V-Sign™-érzékelő 2 lt Jutiklis V-Sign™ 2 lv Sensors V-Sign™ 2 nl V-Sign™-sensor 2 no V-Sign™-sensor 2 pl Czujnik V-Sign™ 2 ro Senzor V-Sign™ 2 ru Датчик V-Sign™ sensor 2 sh V-Sign™ senzor 2 sv V-Sign™ sensor 2	IPX7  0123
OV-A/P/N * PCO₂, PO₂ SpO₂/PR 	en OxiVen™ Sensor de Sensor OxiVen™ es Sensor OxiVen™ fr Capteur OxiVen™ pt Sensor OxiVen™ it Sensore OxiVen™ ja OxiVen™ センサ ar ""OxiVen™ مستشعر bg Сензор OxiVen™ cs Senzor OxiVen™ da OxiVen™-sensor el Αισθητήρας OxiVen™ et OxiVen™-i andur fi OxiVen™-sensori hu OxiVen™-érzékelő lt Jutiklis OxiVen™ lv Sensors OxiVen™ nl OxiVen™-Sensor no OxiVen™-sensor pl Czujnik OxiVen™ ro Senzor OxiVen™ ru Датчик OxiVen™ sh OxiVen™ senzor sv OxiVen™ sensor	IPX7  0123
NON-ADHESIVE WRAP 	en Non-Adhesive Wrap de Klattbandapplikator es Cinta no adhesiva fr Sangle non adhésive pt Faixa não adesiva it Fascia non adesiva ja 非粘着式バンド ar حزام غير لاصق bg Неприважна лента cs Nepřilnavý pásek da Ikke-klebebande bånd el Μη κολλητική ταινία πρόσδεσης et Mittekleepuv kinnituspael fi Tarttumaton kääre hu Tépozárás pánt lt Nelipnus vyūras lv Nelipša lente nl Niet-klevende manchet no Ikke-adhesivt omslag pl Opaska nieprzyklepna ro Bandaj neadeziv ru Неприклеивная лента sh Neprilnajuća traka sv Tejpfritt förband	 0123
GEL-SD * 	en Contact Gel de Kontaktgel es Gel de contacto fr Gel de contact pt Gel de contato it Gel di Contatto ja コンタクトジェル ar جل موصّل bg Контактен гел cs Kontaktgel da Kontaktgel el Γέλη επαφής et Kontaktgeel fi Kontaktigeeli hu Kontaktgel lt Kontaktinis gelis lv Kontaktzeleja nl Contactgel no Kontaktgel pl Żel przewodzący ro Gel de contact ru Контактный гель sh Kontaktini gel sv Kontaktgel en Composition de Zusammensetzung es Componentes fr Constituants pt Componentes it Componenti ja コンポーネント ar المكونات bg Компоненти cs Součásti da Komponenter el Συστατικά στοιχεία et Osad fi Osat hu Komponensek lt Komponentai lv Sastāvdaļas nl Componenten no Komponenter pl Składniki ro Componente ru Компоненты sh Komponente sv Komponenter	 0123
GEL-04 * 		 0123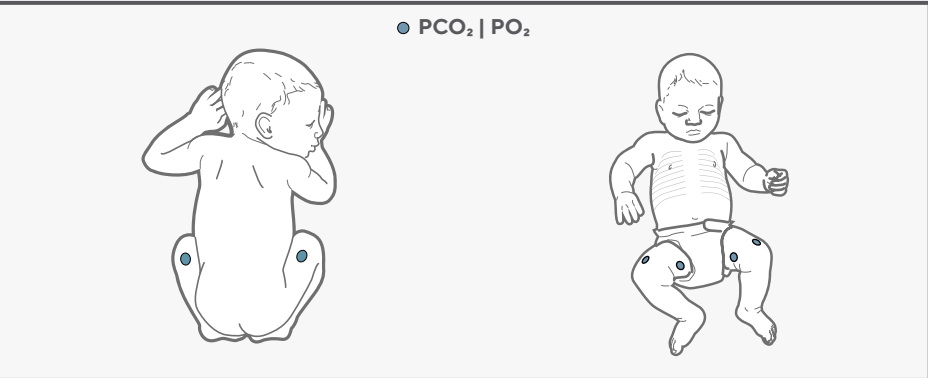

en Please consult the Instruction Manual of the Sentec Digital Monitoring System (SDM/ tCOM+) for complete descriptions of warnings/ cautions, recommendations of temperature setting and site time combinations, and additional information.

Warnings/Cautions Ensure that the Non-Adhesive Wrap is correctly measured around the thigh to avoid a fit that is too loose or tight. Incorrect measurement can cause inaccurate tPCO₂ readings. Avoid overlap between the two ends of the wrap and skin contact with the closure tab. Check the site after sensor application to ensure adequate circulation. A circulation check should be included with each assessment. Avoid damage to the sensor caused by sharp objects or fingernails. Do not use sensor attachment accessories

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or Contact Gel on injured skin or on patients who exhibit allergic reactions. The use of temperatures higher than 41° C requires special attention to patients with susceptible skin, e.g. neonates. A flat, well-perfused area of intact skin is needed. Do not place over large superficial veins or areas of skin breakdown or edema. Do not use product for longer than 24 hours. Do not re-attach the sensor to the same measurement site if any skin irritations are noted. Do not remove the membrane of the sensor for cleaning and disinfection. Use 70% Isopropanol (for external use) or approved cleaning agent listed on [sentec.com/ifu](https://www.sentec.com/ifu). Do not clean the sensor in an ultrasonic cleaner. Do not sterilize any parts described herein. Do not immerse the connector of the sensor cable in any liquid solution.

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* The components listed above do not necessarily correspond to the scope of delivery.