tCOM+ PATIENT MONITOR

Directions for Lay Users

en | da | de | es | fi | fr | it | ja | nl | no | pl | pt | ru | sv

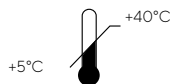
sentec.

tCOM+

Operation Temperature

EN tCOM+ /
Docking Station
DA tCOM+ /
Docking station
DE tCOM+ /
Docking Station
ES tCOM+ /
Estación de calibración
FI tCOM+ /
Telakka-asema
FR tCOM+ /
Station d'accueil
IT tCOM+ /
Docking Station

JA tCOM+ /
ドッキングステーション
NL tCOM+ /
Docking Station
NO tCOM+ /
Dokkingstasjon
PL tCOM+ /
Stacja dokująca
PT tCOM+ /
Docking Station
RU tCOM+ /
док-станция
SV tCOM+ /
Dockningsstation


VS-A/P/N

EN V-Sign™ Sensor 2 ("Sensor")
DA V-Sign™-sensor 2 ("sensor")
DE V-Sign™ Sensor 2 („Sensor“)
ES Sensor V-Sign™ 2 ("Sensor")
FI V-Sign™-sensori 2 ("sensori")
FR Capteur V-Sign™ 2 (« Capteur »)
IT Sensore V-Sign™ 2 ("Sensore")

JA V-Sign™センサ2 (「センサ」)
NL V-Sign™ Sensor 2 („Sensor“)
NO V-Sign™-sensor 2 ("sensor")
PL Czujnik 2 V-Sign™ („czujnik“)
PT Sensor V-Sign™ 2 ("sensor")
RU Датчик V-Sign™ 2 ("sensor")
SV V-Sign™ Sensor 2 ("sensor")

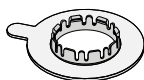
MARe - MI/SF

EN Multi-Site Attachment Ring
DA Fikseringsring
DE Befestigungsring
ES Anillo de fijación multizona
FI Kiinnitysrengas
FR Anneaux de fixation
IT Anello adesivo

JA マルチサイト・
アタッチメントリング
NL Bevestigingsring
NO Festering
PL Pierścień mocujący
PT Anel de fixação multilocalização
RU Крепежное кольцо
SV Multi-Site fixeringsring

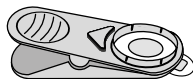
EC-MI

EN Ear Clip
DA Øreclip
DE Ohrclip
ES Clip de oreja
FI Korvaklipsi
FR Pince d'oreille
IT Clip Auricolare

JA ヤークリップ
NL Oorclip
NO Øreklips
PL Klips nauszny
PT Clipe de orelha
RU Ушной зажим
SV Öronklämma

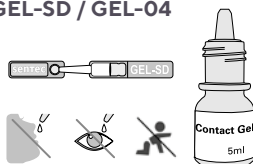GEL-SD / GEL-04


EN Contact Gel
DA Kontaktgel
DE Kontaktgel
ES Gel de contacto
FI Kontaktigeeli
FR Gel de contact
IT Gel di Contatto

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
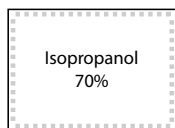
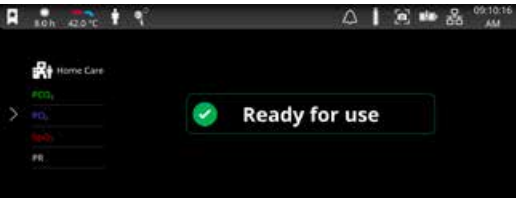
ENSURE tCOM+ DISPLAYS
"READY FOR USE".

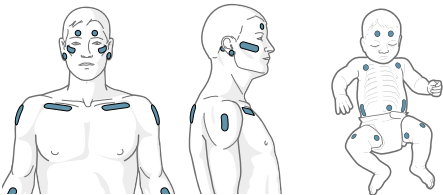MEASUREMENT SITES

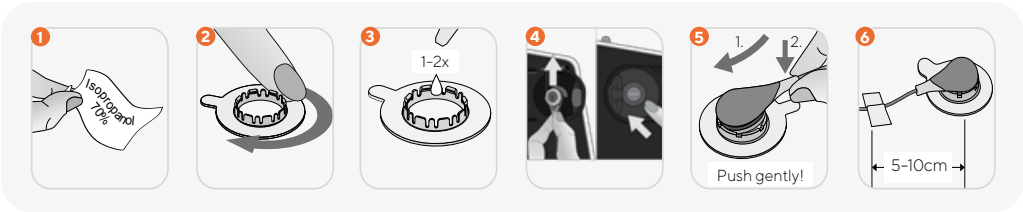Note: The homecare provider is responsible for selecting the appropriate measurement site(s).

- 1 Clean skin at Measurement Site selected by the home care provider, using 70 % Isopropanol wipe. Let the site dry.
- 2 Pull adhesive tape off Multi-Site Attachment Ring (MARE) and attach MARE to measurement site, pressing gently around the ring.
- 3 Apply 1-2 drops of Contact Gel to the skin area in the center of the ring.
Note: Avoid contact with injured skin. Do not use when exhibiting allergic reactions to one of the components.

- 4 Open Docking Station, remove sensor, close Docking Station.
Note: Do not touch the sensor surface with any hard objects or fingernails. Otherwise, the sensor could be damaged.
- 5 Click sensor into MARE and rotate sensor to spread Contact Gel.
- 6 Secure cable with an adhesive tape.
Important Note: Route the sensor cable properly (i.e., with a Clothing Clip) to avoid entanglement or strangulation.
Caution: Do not place any tape over the sensor head!

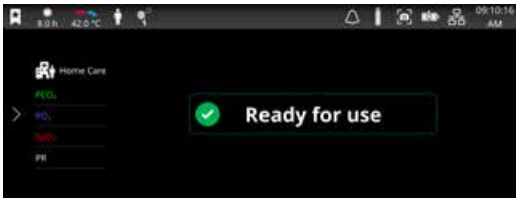
If you need to interrupt a session after monitoring has started (e.g., to go to the bathroom), you may disconnect the sensor for up to 20 minutes.

20 min


If you disconnect the sensor for more than 30 minutes, the recording will be lost.

Tap anywhere on the screen to activate it and/or to access a tutorial on how to apply the Multi-Site Attachment Ring.


B) Sensor application using EC-MI

ENSURE tCOM+ DISPLAYS
"READY FOR USE".

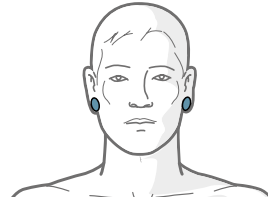
MEASUREMENT SITES

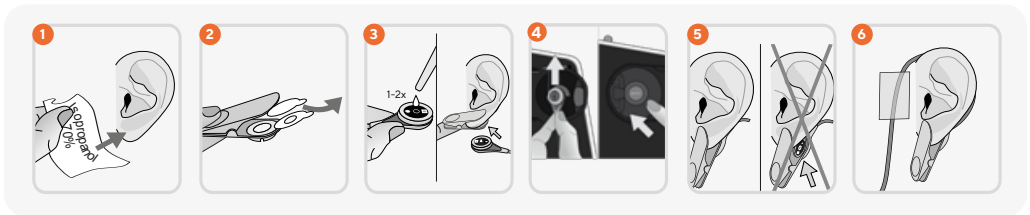- 1 Clean skin at Measurement Site selected by the home care provider, using 70 % Isopropanol wipe.
- 2 Pull both adhesive tapes off Ear Clip, attach Ear Clip to earlobe with retainer ring on the back side. Verify that the skin under the retainer ring's adhesive is not wrinkled and that the hole in the center of the retainer ring completely covers the skin.
- 3 Apply 1-2 drops of Contact Gel to the center of the sensor surface.
Note: Avoid contact with injured skin. Do not use when exhibiting allergic reactions to one of the components.
- 4 Open Docking Station, remove sensor, close Docking Station.
Note: Do not touch the sensor surface with any hard objects or fingernails. Otherwise, the sensor could be damaged.
- 5 Click sensor into Ear Clip, rotate sensor to spread Contact Gel.
Note: Earlobe must cover the entire sensor surface. Be sure to hold the sensor such that the contact liquid does not run off the sensor face.
- 6 Wrap the sensor cable around the ear and secure it with an adhesive tape.
Note: Do not lie on Ear Clip as this may lead to incorrect measurements.
Important Note: Route the sensor cable properly (i.e., with a Clothing Clip) to avoid entanglement or strangulation.

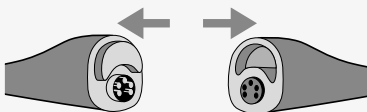
If you need to interrupt a session after monitoring has started (e.g., to go to the bathroom), you may disconnect the sensor for up to 20 minutes.

If you disconnect the sensor for more than 30 minutes, the recording will be lost.

Tap anywhere on the screen to activate it and/or to access a tutorial on how to apply the Multi-Site Attachment Ring.

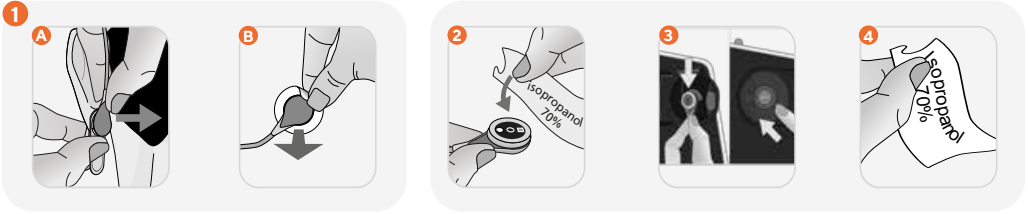C) Sensor Removal

- 1 When monitoring is completed, remove **4** Ear Clip or gently peel off **3** Multi-Site Attachment Ring, then remove sensor from retainer ring.
- 2 Clean sensor and cable with Isopropanol 70 % soaked wipe.
- 3 Open Docking Station, put sensor into cradle inside the door with the red light visible, close Docking Station.
Note: Do not turn off the device.
- 4 Clean measurement site with Isopropanol 70 % wipe. Dispose of Ear Clip or Multi-Site Attachment Ring in the regular garbage.
Note: Your home care provider will ensure professional and hygienic maintenance of the system once returned.

⚠ IMPORTANT: Do not turn off the device at the end of the measurement. The evaluation and analysis of the measurements is carried out by the home care provider. Analysis of data is done by the supervising medical doctor.

⚠ IMPORTANT

- The device may only be installed by a home care provider's instructed person.
- In the event of unforeseen incidents, error messages, unexplained changes in the performance of the device, or if you require any other assistance, please contact your home care provider's instructed person. If in doubt, you can interrupt the measurement by removing the sensor, cleaning it, and placing it in the Docking Station.
- Solely use the device as per this Lay User Manual. Do not make any changes to the device settings. Sentec is not responsible for any injuries or damages incurred due to improper use of the device contradictory to this Lay User Manual.
- Do not make any changes to the mains plug of the device and secure the mains cable against squeezing or bending.
- Keep electronic devices and cables out of reach of children and pets and supervise them during monitoring.
- Make sure that children do not swallow Contact Gel or small parts, such as the sensor, the Multi-Site Attachment Rings, or Ear Clip.
- Only use Sentec Contact Gel. Do not use any other contact gel/liquid. Do not use Sentec Contact Gel on any open wound, do not swallow it or apply it to the eyes.
- It is not recommended to use the Contact Gel on patients who exhibit allergic reactions to any of the components.
- Temporary skin irritations or reddening could occur during monitoring at increased sensor temperatures (i.e., above the normal body surface temperature). The skin reddening will generally disappear within a few hours to max. a few days without any side effects or scars. The likelihood of these occurrences depends on patient skin condition, sensor temperature and exposure time.
- Do not reattach the sensor to the same site if any skin irritations are noted during site inspection.
- It is not recommended to use sensor attachment accessories on patients who exhibit allergic reactions to adhesive tapes.
- Application of any pressure to the measurement site (e.g., by using a pressure bandage) may cause pressure insufficient blood supply of tissue at the measurement site and, consequently, inaccurate measurements, skin/tissue damage or – in combination with heated sensors – burns.
- The Multi-Site Attachment Rings and Ear Clip are for single use.
- Do not pour any liquid on the tCOM+, its accessories, connectors, switches, or openings in the chassis. If the tCOM+ has been wetted accidentally, it must be removed from AC power, wiped dry externally, allowed to dry thoroughly, and inspected by qualified service personnel before further use. The tCOM+ is protected against vertically falling drops of water.
- The tCOM+ may not be installed and operated in moist rooms (e.g., bathroom).
- Keep the device away from heat sources (e.g., heating) and at least 50 cm from flammable materials.
- Mobile telephones and other radio communication devices must be kept 1 m away from the device.
- Do not cover the device.
- Do not use a non-invasive blood pressure cuff or other constricting devices on the same limb as the sensor. This may result in inaccurate measurements.
- Protect all devices from lint, dust, and direct sunlight.
- High ambient light conditions may result in inaccurate measurements.

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Sentec AG