

## IPV® 1 Compatible Cleaning and Disinfection Solutions

The provided cleaning and disinfection solutions have been evaluated based on their chemical compatibility with the materials found in the following device(s). The device has been assessed for chemical compatibility of material only and not cleaning/disinfection efficacy.

Product Name	Referred to as	REF
IPV® 1	Control unit / Device	S00065 / S00065-EU

The following list of name brand products for cleaning and disinfection may be substituted by other name brand products of equivalent composition. However, as there are many factors (contact time, force applied during wiping, etc.) that can influence the functionality of the device, it is the responsibility of the user to test the functionality of the device after cleaning and/or disinfection.

**Note:** For specific device use, handling, storage, and recommended cleaning and/or disinfection protocols, refer to the device specific user manual found at [sentec.com/manuals/](https://www.sentec.com/manuals/)

Table 1: Surface Cleaning and Low-level Disinfection		
Name Brand / Manufacturer	Composition	Recommended Procedure
CaviWipes™	Isopropyl Alcohol Butyl Cellosolve Benzethonium Chloride (quaternary ammonium compound)	Carefully wipe all the surfaces of the device.  Keep damp for 1 minute.  Allow to dry.
Super Sani-Cloth®	Isopropyl Alcohol  Quaternary ammonium compounds, C12-18-alkyl [(ethylphenyl) methyl] dimethyl, chlorides  n-Alkyl dimethyl Benzyl ammonium chloride	Carefully wipe all the surfaces of the device.  Keep damp for 1 minute.  Allow to dry.

Table 2: Intermediate-level Disinfection		
Name Brand / Manufacturer	Composition	Recommended Procedure
Super Sani-Cloth®	Isopropyl Alcohol  Quaternary ammonium compounds, C12-18-alkyl [(ethylphenyl) methyl] dimethyl, chlorides  n-Alkyl dimethyl Benzyl ammonium chloride	Carefully wipe all the surfaces of the device.  Keep damp for 1 minute.  Allow to dry.
Isopropanol*	70% Isopropanol*	Carefully wipe all the surfaces of the device.  Keep damp for 1 minute.  Allow to dry.

\*70% Isopropanol must only be used on pre-cleaned surfaces. Use after cleaning with CaviWipes™ or Sani-Cloth® when appropriate.

Table 3: High-level Disinfection		
Name Brand / Manufacturer	Composition	Recommended Procedure
N/A	N/A	N/A



Percussionaire® Corporation  
130 McGhee Rd, Ste. 109  
Sandpoint, ID 83864 USA

CE  
0123



[sentec.com/manuals/](https://sentec.com/manuals/)

PAC-ENG-LIT-778 / Rev.0  
Date of Release: 2025-06