
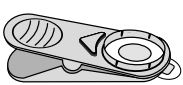
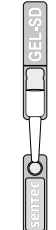
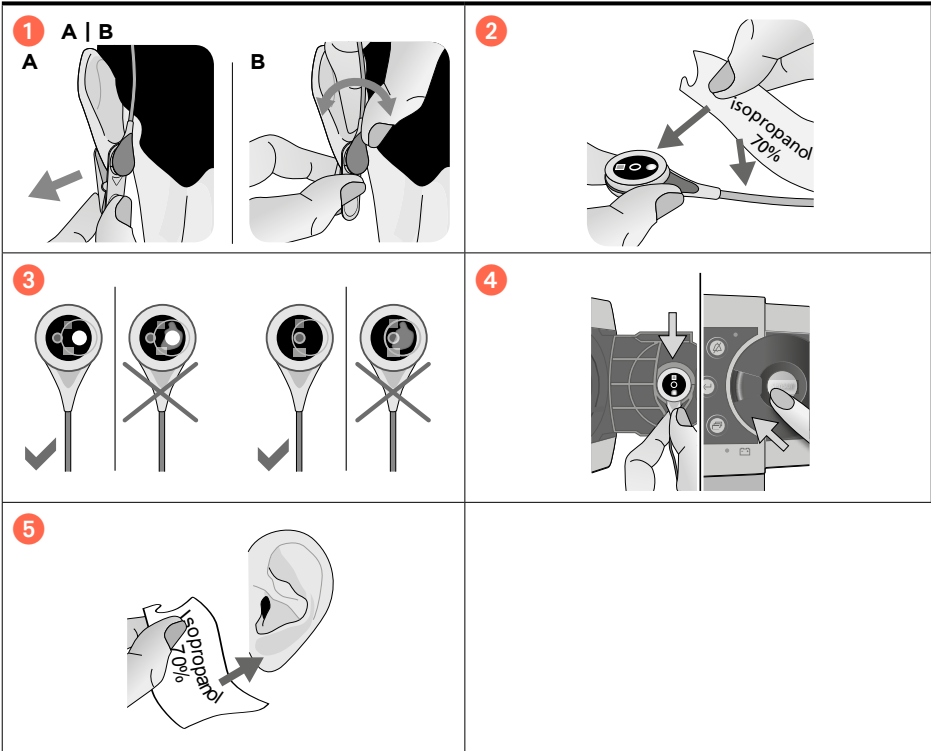
VS-A/P/N PCO2, SpO2/PR 	en V-Sign™ Sensor 2 de V-Sign™ Sensor 2 es Sensor V-Sign™ 2 fr Capteur V-Sign™ 2 pt Sensor V-Sign™ 2 it Sensore V-Sign™ 2 zh V-Sign™ 传感器 2 ja V-Sign™ センサ 2 ar V-Sign™ 2 مقياس bg Сензор V-Sign™ 2 cs Sensor V-Sign™ 2 da V-Sign™-sensoren 2 el Αισθητήρας V-Sign™ 2 et V-Sign™-i andur 2 fi V-Sign™-sensori 2 hu V-Sign™-érzékelő 2 it Jutiklis V-Sign™ 2 lv Sensors V-Sign™ 2 nl V-Sign™ Sensor 2 no V-Sign™-sensor 2 pl Czujnik 2 V-Sign™ ro Senzor V-Sign™ 2 ru Датчик 2 V-Sign™ sh V-Sign™ sensor 2 sv V-Sign™ sensor 2	IPX7 +15°C +26°C    0123
OV-A/P/N PCO2, PO2, SpO2/PR 	en OxiVen™ Sensor de OxiVen™ Sensor es Sensor OxiVen™ fr Capteur OxiVen™ pt Sensor OxiVen™ it Sensore OxiVen™ zh OxiVen™ 传感器 ja OxiVen™ センサ ar OxiVen™ مقياس bg Сензор OxiVen™ cs Senzor OxiVen™ da OxiVen™-sensor el Αισθητήρας OxiVen™ et OxiVen™-i andur fi OxiVen™-sensori hu OxiVen™-érzékelő it Jutiklis OxiVen™ lv Sensors OxiVen™ nl OxiVen™ Sensor no OxiVen™-sensor pl Czujnik OxiVen™ ro Senzor OxiVen™ ru Датчик OxiVen™ sh OxiVen™ sensor sv OxiVen™ sensor	IPX7 +15°C +26°C    0123
EC-MI 	en Ear Clip for mature, intact skin de Ohrclip für normale, gesunde Haut es Clip de oreja para piel madura e intacta fr Pince d'oreille pour les peaux intactes pt Clip de Orelha para pele madura e intacta it Clip Auricolare per pelli mature e intatte zh 耳夹 適合正常、健康的皮肤 ja イーアークリップ 成熟肌、無傷な肌用 ar شبك الأذن للجلد السليم المتقدمة في السن bg Клип за ухо за зряла, здрава кожа cs Klip na ucho pro zralou, nepoškozenou pokožku da Øreclip til moden, ubeskadiget hud el Κλίπ αυτίου για ώριμο, άβλαβό δέρμα et Kõrvaklamber normaalse, terve naha jaoks fi Korvaklipsi ehjälle iholle hu Füleszűrtető éretl, ép bőrre lv Ausies spauštukas normaliai, sveikai odai lt Ausis klipsis nobriedusiai adai bez bojajumiem nl Øreclip voor volgroeiende, intacte huid no Øreklips til moden, intakt hud pl Klips nauszny do skóry dojrzalej, nienaruszonej ro Clip de urechi pentru pielea matură și intactă ru Ушная клипса для зрелой, неповрежденной кожи sh Klips za uvo za zrelo, neobesedenu kožu sv Öronklämma för mogen, intakt hud	+10°C +30° 60% 25%  
GEL-SD 	en Contact Gel de Kontaktgel es Gel de contacto fr Gel de contact pt Gel de contato it Gel di Contatto zh 接觸凝膠 ja コンタクトジェル ar جل موشل bg Контактен гел cs Kontaktní gel da Kontaktgel el Γέλ επώδης et Kontaktgeel fi Kontaktigeeli hu Kontaktgél lt Kontaktinis gelis lv Kontaktželeja nl Contactgel no Kontaktgel pl Żel przewodzący ro Gel de contact ru Контактный гель sh Kontaktni gel sv Kontaktgel	+10°C +30°C  
GEL-04 	en Contact Gel de Kontaktgel es Gel de contacto fr Gel de contact pt Gel de contato it Gel di Contatto zh 接觸凝膠 ja コンタクトジェル ar جل موشل bg Контактен гел cs Kontaktní gel da Kontaktgel el Γέλ επώδης et Kontaktgeel fi Kontaktigeeli hu Kontaktgél lt Kontaktinis gelis lv Kontaktželeja nl Contactgel no Kontaktgel pl Żel przewodzący ro Gel de contact ru Контактный гель sh Kontaktni gel sv Kontaktgel	+10°C +30°C  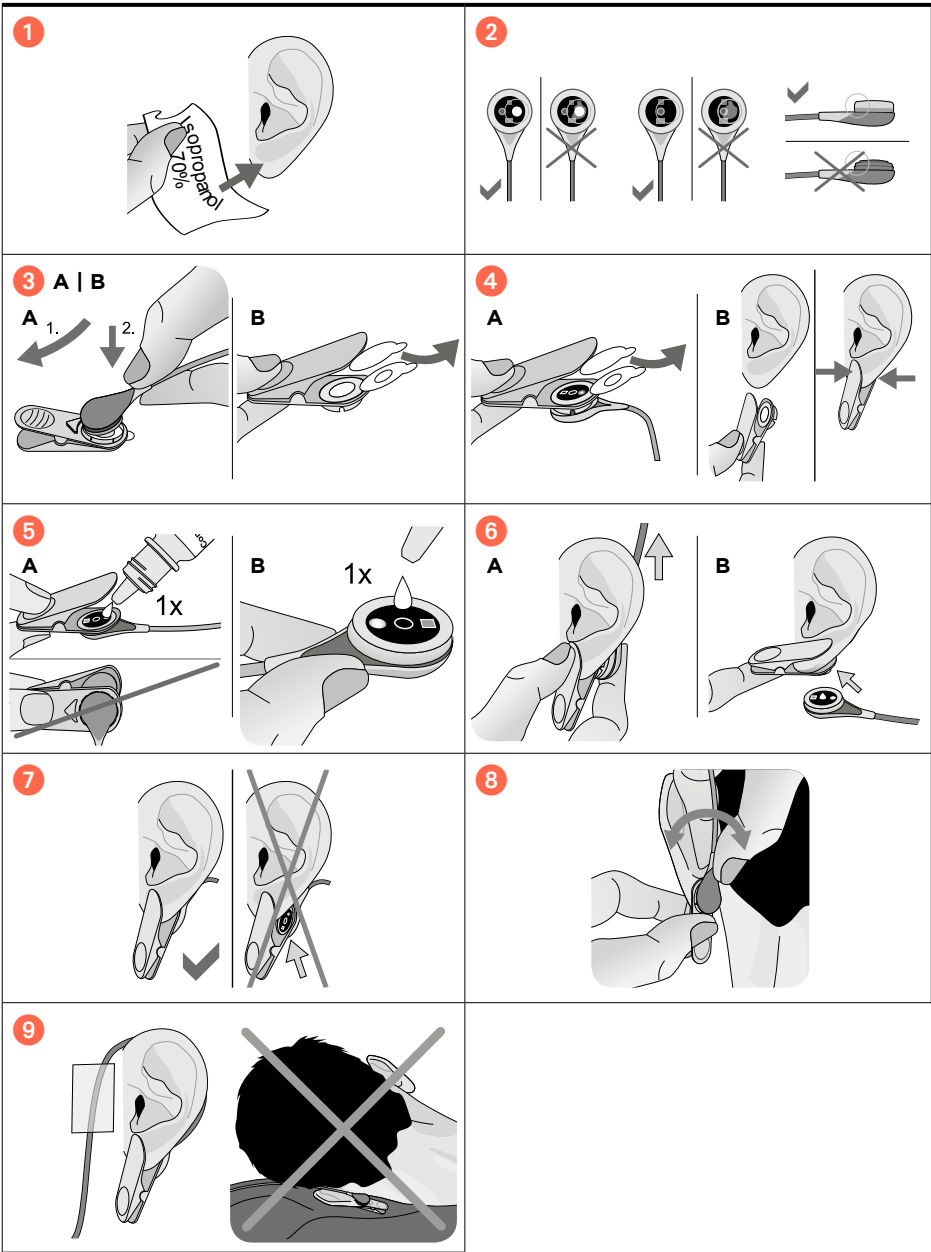

en Please consult the Instruction Manual of the SenTec Digital Monitoring System for complete descriptions of warnings/cautions, recommendations of temperature setting and site time combinations, and additional information.

Contraindications

The use of the Ear Clip is contraindicated in case of:

- patients whose earlobes are too small to ensure adequate sensor application (e.g. neonates).
- conditions where the patient is in the Trendelenburg's position (head lower than the heart).

Warnings/Cautions

The use of temperatures higher than 41 °C requires special attention to patients with susceptible skin, e.g. neonates, geriatric patients, burn victims, patients with skin diseases. Ensure that all parts are applied correctly and physically intact and that the sensor is not damaged by sharp-edged objects or fingernails. Incorrect application and damage can cause incorrect measurements. Always apply sensor and Contact Gel on intact and closed skin sites. Pierced earlobes may result in incorrect

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tcPCO2 and/or tcPO2 measurements. Do not apply on patients who exhibit allergic reactions to these disposable. Do not use the Ear Clip for sensor attachment on any other site than the earlobe. Do not excessively rotate the sensor within the Ear Clip after sensor attachment. This may cause incorrect measurements. Do not change the opening angle of the Ear Clip after sensor application. This may cause incorrect measurements. Do not re-attach the sensor to the same measurement site if any skin irritations are noted. Discard the Ear Clip in case of defects or loss of spring tension sufficient to allow slippage or movement of the sensor from its proper position on the earlobe. Do not remove the membrane of the sensor for cleaning and disinfection. Use 70% Isopropanol or approved cleaning agent listed on www.senotec.com/ifu. Do not clean the sensor in an ultrasonic cleaner. Do not sterilize any parts described herein (e.g. by irradiation, steam, ethylene oxide or plasma method). Do not immerse the connector of the sensor cable in any liquid solution.

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SenTec AG