


Quick Guide for Installation at Home User Site



- Select SDM “Home Profile” in V-STATS™
- Couple Isolation Transformer and mains cable plugs
- Connect Isolation Transformer with SDM
- Plug mains cable into socket

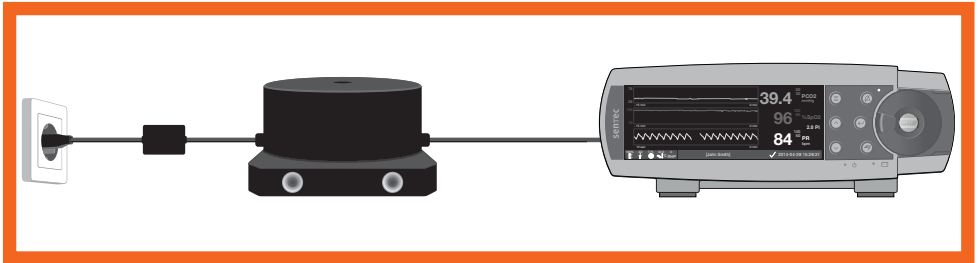

Refer to “SDMS - Installation at Home User Site by Instructed Personnel” (HB-010103) for detailed information.



www.sentec.com/ifu



SenTec AG
Ringstrasse 39
CH-4106 Therwil
Switzerland
www.sentec.com

HB-011028-b
Date of release: 11/2020



CE
0123

Care with
Confidence